

Confidence for 12hrs

using the Multi-Patient kit



medRAD® Stellant MP

The Multi-Patient System for your MEDRAD® Stellant

Perform multiple studies with a dedicated MP syringe system – for up to 12 hours.



1

Multi-Guard

A recessed connection protects the fluid path from accidental contamination.

2

Tubing Design

The MEDRAD® Stellant multi-patient system protects patients with a 15 cm multi-patient tube, that connects to the syringes and is out of patients' reach. The corresponding single-patient tube is 250 cm long and disposed of after each study.

3

Check Valves

Two check valves on the single-patient tube prevents backflow from the patient.

Multi-Patient Spike

Spiques can be used to access multiple containers of contrast or saline for up to 12 hours.

MEDRAD® Stellant enables the flexibility of injecting contrast and saline, while also offering DualFlow (simultaneous injection of contrast and saline in user-selected ratios) – which offers many clinical and workflow advantages, such as:
Using saline to perform a patency check, reducing image artifacts and providing brighter arterial imaging in CT.^{1,2}

Clear Direction.

Boost Your Image.


¹ Kerl JM, et al. Radiology. 2008;247(2):356–364.

² Jensen, CT. NASCI Abstract # 05-A-69-NASCI.

Technical Data for MEDRAD® Stellant Multi-Patient Kit and Single-Patient Disposable


Multi-Patient Kit

Catalog Number: SDS MP1

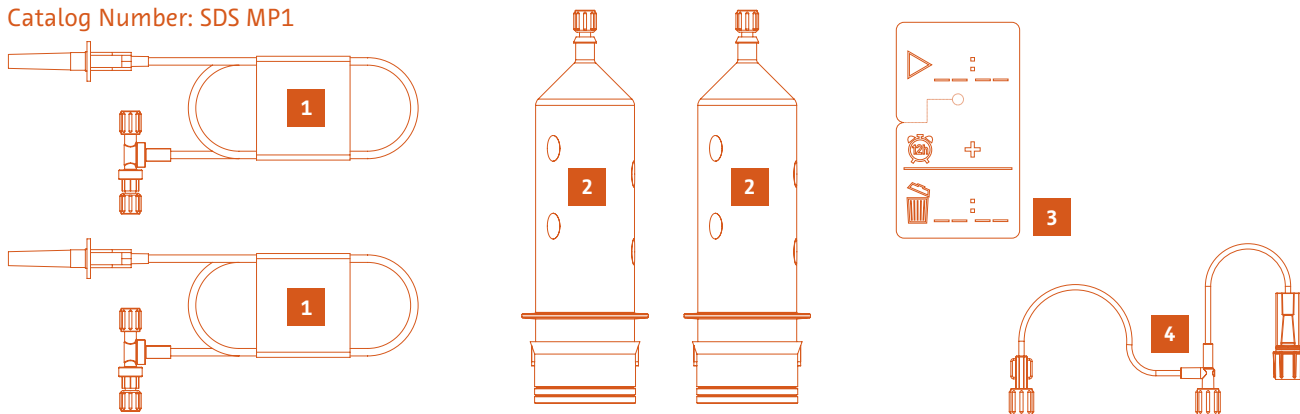

Single-Patient Disposable

Catalog Number: SPD 250

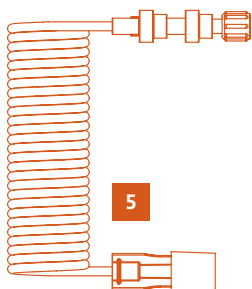

Item	Component	Material	Other Information
1	2 Multi-Patient Transfer Sets (MPTS)	Polyvinyl Chloride (PVC), Acrylonitrile butadiene styrene (ABS), Polystyrene (PS), Polycarbonate (PC), High Density Polyethylene (HDPE), Polypropylene (PP), Silicone Rubber	Use life: 12 hours Length: 115 cm Two check valves per MPTS
2	2 Syringe Assemblies (includes plunger)	Polyester (PET), PC, PP, Santoprene, Silicone	Use life: 12 hours Capacity: 200 mL
3	Expiration Label	Coated Vinyl	None
4	Multi-Patient Tubing Set (MPTS)	PVC, PC, PP	Use life: 12 hours
5	Single-Patient Tubing Set (SPTS)	PVC, Polycarbonate, Silicone Rubber, Silicone	Use life: 1 patient Length: 250 cm when fully extended Two check valves

General information

Shelf Life: 4 years	Not made with Natural Rubber Latex
Sterilisation Method: Radiation	SDS MP1 Primary Packaging: Polystyrene with Tyvek® cover
Pressure rated to 350 psi	SPD 250 Primary Packaging: Polyethylene with Tyvek® cover
Di(2-ethylhexyl)phthalate (DEHP) Free	Secondary Packaging: Cardboard Shipper
Non-Pyrogenic fluid path	

Bayer reserves the right to modify the specifications and features described herein or to discontinue any product or service identified in this publication at any time without prior notice or obligation. Please contact your authorised Bayer representative for the most current information. All patient data that appear in this document are fictitious. No actual patient information is shown.

Bayer, the Bayer Cross, Certegra, Certegra Workstation, MEDRAD, MEDRAD FluiDot and MEDRAD Stellant are trademarks owned by and/or registered to Bayer in the U.S. and/or other countries. Other trademarks and company names mentioned herein are properties of their respective owners and are used herein solely for informational purposes. No relationship or endorsement should be inferred or implied.

©2020 Bayer. This material may not be reproduced, displayed, modified or distributed without the express prior written consent of Bayer.



Bayer HealthCare LLC
100 Bayer Boulevard
P.O. Box 915
Whippany, NJ 07981
U.S.A.
Phone: +1-412-767-2400
+1-800-633-7231
Fax: +1-412-767-4120



Manufacturer
Bayer Medical Care Inc.
1 Bayer Drive
Indianola, PA 15051-0780
U.S.A.
Phone: +1-412-767-2400
+1-800-633-7231
Fax: +1-412-767-4120

Authorised Australian Representative
Imaxeon Pty Ltd
Unit 1, 38 – 46 South Street
Rydalmere, NSW 2116
Australia
Phone: 02 8845 4999
Fax: 02 8845 4998
Customer Service: 1800 633 723

More information on
radiologysolutions.bayer.com